**Verify<sup>®</sup>**

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contiene 1 test

# **Technical File**

## **Total Cholesterol Test Card (Whole Blood)**

Document No.:

**HANGZHOU ALLTEST BIOTECH CO., LTD**

Address: # 550, Yinhai Street, Hangzhou Economic & Technological Development Area,

Hangzhou-310018, P.R. China

# CATALOGUE

<b>No.</b>	<b>Content</b>	<b>Page</b>
<b>1</b>	<b>GENERAL DESCRIPTION</b>	<b>3</b>
1.1	INTENDED USE	3
1.2	SUMMARY	3
1.3	PRINCIPLE	3
1.4	REAGENT	3
1.5	PRECAUTIONS	3
1.6	STORAGE AND STABILITY	3
1.7	SPECIMEN COLLECTION AND PREPARATION	3
1.8	STANDARD TESTING PROCEDURE	4
1.9	INTERPRETATION OF RESULTS	4
1.10	COMPOSITION OF PRODUCT	4
<b>2</b>	<b>TECHNICAL REPORT FOR URINARY TRACT INFECTIONS TEST</b>	<b>1</b>
2.1	MATERIALS	1
2.2	REPEATABILITY	1
2.3	CLINICAL TEST	2
2.4	INTERFERING SUBSTANCE STUDY	2
2.5	ACCURACY	4
2.6	TIME FLEX	4
2.7	OPEN POUCH STABILITY	5
2.8	ACCELERATE STABILITY	7
2.9	REAL TIME STABILITY	9
2.10	SAMPLE SIZE TEST	10
	DOCUMENT HISTORY SUMMARY	11

## 1 General description

### 1.1 Intended Use

Suitable for Semi-quantitative detection of Total Cholesterol in human whole blood.

### 1.2 Summary

Cholesterol is a lipid that is an important component of cell membranes, hormones, and vitamin D, and is a risk factor for cardiovascular disease. Through blood tests, we can know a person's total cholesterol level so that we can manage our health.

### 1.3 Principle

The tested sample is added to the sample adding area of the test card, and it infiltrates evenly and rapidly. During the infiltration process, the blood cells are filtered, and the plasma diffuses to the reaction later and the enzymes and chemicals in the reaction layer. The product reacts and develops color, and compare with the color chart for eye reading judgment.

### 1.4 Reagent

Based on the dry weight at the time of impregnation, the concentrations given may vary within manufacturing tolerances.

Reagent	Composition
Total Cholesterol(TC)	Cholesterol Esterase > 0.3U
	Cholesterol Oxidase > 0.16U
	4-Aminoantipyrine > 0.06mg

### 1.5 Precautions

- For self-testing in vitro diagnostic use only.
- The card should remain in the closed packaging bag until use.
- Do not use after the expiration date.
- Do not touch the sample area of the card.
- Discard any discolored or damaged card.
- The used card should be discarded according to local regulations after testing.

### 1.6 Storage and Stability

Store as packaged in the closed packaging bag, either at room temperature or refrigerated (2-30 °C). Keep out of direct sunlight. Cards are stable through the expiration date printed on the package. DO NOT FREEZE. Do not use beyond the expiration date.


Note: Stability may be reduced in high humidity conditions.

### 1.7 Specimen collection and preparation

1. Acceptable specimens include capillary whole blood from the fingertip.
2. Whole blood specimens should be tested immediately after lancing.
3. Fingertip sample must be obtained from a free spontaneous blood flow after the puncture.
4. Do not collect capillary blood by pressing or squeezing the finger too hard as this will push fluid from the tissue into the blood and may cause false results.

## 1.8 Standard testing procedure

1. Wash your hands with soap and rinse with clear warm water.
2. Bring the pouch to room temperature before opening it. Open the pouch, remove the test card and place it on a clean and level surface. Run the test within one hour and best results will be obtained if the test is performed immediately after opening the foil pouch. Remove the dropper, lancet and alcohol pad, place them close to the test card.
3. Carefully pull off and dispose the released cap of the lancet.
4. Use the provided alcohol pad to clean the fingertip of the middle or ring finger as the puncture site. Allow to air dry.
5. Press the lancet, on the side from where the cap was extracted; the tip retracts automatically and safely after use. Massage the hand without touching the puncture site by massaging the hand towards the fingertip of the middle or ring finger to be punctured.
6. Keeping the hand down massage the end that was pricked to obtain a blood drop.
7. Without squeezing the capillary dropper bulb, put it in contact with the blood. The blood migrates into the to the line indicated on the capillary dropper. You may massage again your finger to obtain more blood if the blood does not reach the indicated line. Avoid of air bubbles.
8. Release the blood collected into the sample well of the card by squeezing the dropper bulb.
9. Wait for the blood to be totally dispensed in the well and wait 2 minutes before turning the test kit over. DO NOT turn over the test kit before 2 minutes.
10. Turn over the test kit after 2 minutes and rotate the card and match the closest matching color within 30 seconds.
11. The test result must be read within 2 minutes 30 seconds of applying the blood sample.

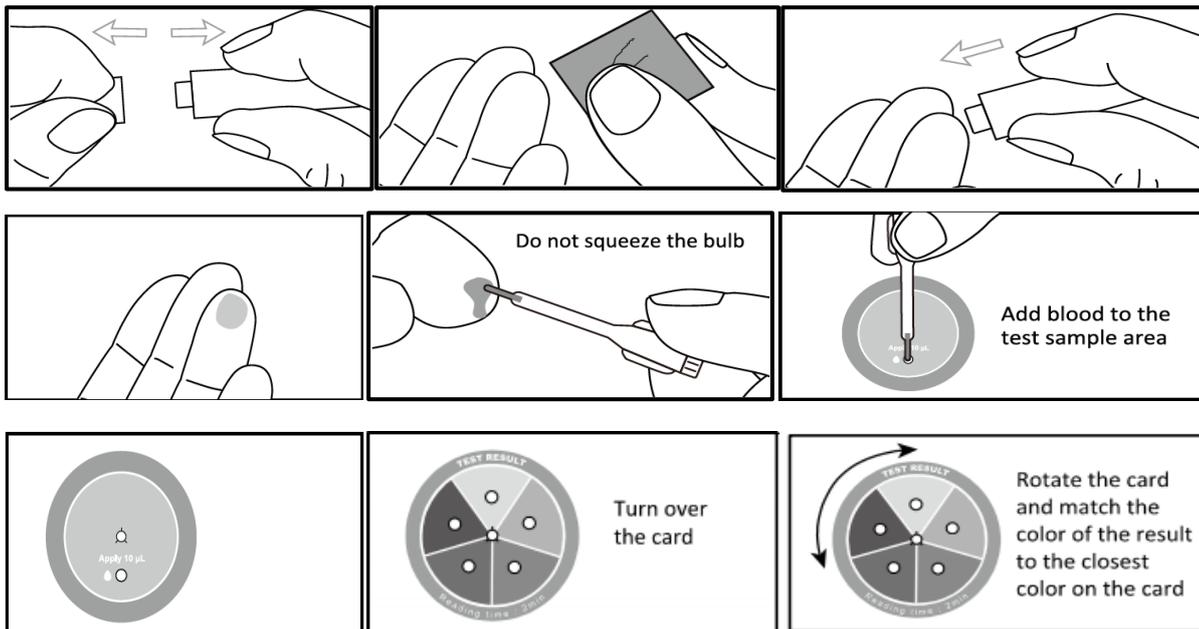## 1.9 Interpretation of results

The color card on the back of the product shows the total cholesterol concentration range corresponding to different colors. By comparing the color of the test result with the color on the color card, the range of total cholesterol concentration where the result is located can be obtained.

### **1.10 Composition of Product**

- Test Card
- Dropper
- Package Insert
- Safety lancet
- Alcohol pad

## 2 Technical Report for Total Cholesterol Test Card (Whole Blood)

### 2.1 Materials:

Total Cholesterol Test Card(Whole Blood) of Hangzhou Alltest Biotechnology Co., Ltd:

Lot1:ATTCH25020001

Lot2: ATTCH25020002

Lot3: ATTCH25020003

### 2.2 Repeatability

#### Method

Three batches of total cholesterol test cards were used. Each batch was tested with five concentration standards respectively, and each concentration was repeated ten times. Read the result by comparing the color reactions of the color blocks.

#### Result

Table: Repeatability Study Results

lot	Specimen	Concentration range(mg/dL)	Is it within the range										
			Test 1	Test 2	Test 3	Test 4	Test 5	Test 6	Test 7	Test 8	Test 9	Test10	
ATTCH25020001	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
ATTCH25020002	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
ATTCH25020003	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes	yes

**Conclusion:**

All the test results on the total cholesterol test card show the correct results. The results show that the total cholesterol test cards have consistent performance and good repeatability.

**2.3 Clinical test****Method**

The total cholesterol test card was compared with the Mindray biochemical analyzer. Fifty-six clinical samples collected from the subjects present were tested. Compare the reagent area with the corresponding color block on the color card at the specified time. Rotate the color card and match carefully. The test results are shown in the following table.

**Results:**

Table: Clinical Study Results

Number of Samples	Concentration range	Alltest strips	the Mindray biochemical analyzer	Whether it is consistent
56	0-99	0	0	yes
	100-149	14	14	yes
	150-199	39	39	yes
	200-259	3	3	yes
	Above 250	0	0	yes

**Conclusion:** A clinical trial was conducted to compare the total cholesterol test card of Hangzhou Alltest Biotechnology Co., Ltd. with the Mindray Biochemical analyzer. 56 samples were analyzed as follows: The test results of the total cholesterol test card of Hangzhou Alltest Biotechnology Co., Ltd. and the test results of Mindray Biochemical analyzer have a 100% coincidence rate, indicating that the two have a high coincidence rate in performance testing.

**2.4 Interfering Substance Study****Method:**

Detect interfering components in whole blood standards. A certain possible interfering substance was added to the whole blood standards with concentrations of 125mg/dL and 225mg/dL respectively, one at a time. The concentrations of the interfering substances are listed in the table below. Each blood sample was tested using five repeated total cholesterol test cards. According to the instructions on the packaging manual, compare the color reaction with the color blocks on the color card and read the result.

**Interfering Substances:**

The following potentially interfering substances at concentrations indicated were added to the Whole blood standard product:

Interfering Substances Tested	Concentration Level
Ascorbic acid	3 mg/dL
Bilirubin	20 mg/dL

Creatinine	30 mg/dL
Hemoglobin	400 mg/dL
Urea	500 mg/dL
EDTA	640 mg/dL
Heparin-Lithium	100 mg/dL
Hematocrit	30~55%

**Results:**

Table: Interfering Substance Study Results

Interference Substance	Interference substances Conc. (mg/dl)	Testing Results			
		125mg/dL Whole blood standard product without spiked interference substance	125mg/dL Whole blood standard product with spiked interference substance	225mg/dL Whole blood standard product without spiked interference substance	225mg/dL Whole blood standard product with spiked interference substance
Ascorbic acid	3 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Bilirubin	20 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Creatinine	30 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Hemoglobin	400 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Urea	500 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
EDTA	640 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Heparin-Lithium	100 mg/dL	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL
Hematocrit	30~55%	100-149mg/dL	100-149mg/dL	200-249mg/dL	200-249mg/dL

**Conclusion:**

From the above data, none of the substances at the concentration levels tested interfered with the ability for the Total Cholesterol Test Card to obtain correct results.

**2.5 Accuracy****Method:**

The accuracy of the total cholesterol test card (whole blood) was tested using samples from three different batches. For each batch, five different concentrations of whole blood standard samples were tested respectively, and the process was repeated five times. Compare the test results of the total cholesterol detection card with those of the Mindray biochemical analyzer. The test was conducted in accordance with the operation procedures stated in the instructions. The results are listed in the table below.

**Results:**

Table: Accuracy Study Results

lot	Specimen	Alltest strips					the Mindray biochemical analyzer	Whether it is consistent
		Test1	Test2	Test3	Test4	Test5		
ATTCH25020001	50mg/dL	0-99	0-99	0-99	0-99	0-99	53.2	yes
	125mg/dL	100-149	100-149	100-149	100-149	100-149	127.5	yes
	175mg/dL	150-199	150-199	150-199	150-199	150-199	178.2	yes
	225mg/dL	200-259	200-259	200-259	200-259	200-259	226.7	yes
	275mg/dL	Above 250	Above 250	Above 250	Above 250	Above 250	271.4	yes
ATTCH25020001	50mg/dL	0-99	0-99	0-99	0-99	0-99	53.2	yes
	125mg/dL	100-149	100-149	100-149	100-149	100-149	127.5	yes
	175mg/dL	150-199	150-199	150-199	150-199	150-199	178.2	yes
	225mg/dL	200-259	200-259	200-259	200-259	200-259	226.7	yes
	275mg/dL	Above 250	Above 250	Above 250	Above 250	Above 250	271.4	yes
ATTCH25020001	50mg/dL	0-99	0-99	0-99	0-99	0-99	53.2	yes
	125mg/dL	100-149	100-149	100-149	100-149	100-149	127.5	yes
	175mg/dL	150-199	150-199	150-199	150-199	150-199	178.2	yes
	225mg/dL	200-259	200-259	200-259	200-259	200-259	226.7	yes
	275mg/dL	Above 250	Above 250	Above 250	Above 250	Above 250	271.4	yes

**Conclusion:**

A comparison was made between the total cholesterol test card of Hangzhou AllTest Biotechnology Co., Ltd. and the Mindray Biochemical Analyzer. The test results of the total cholesterol test card of Hangzhou AllTest Biotechnology Co., Ltd. are relatively consistent with those of Mindray Biochemical Analyzer, indicating that the accuracy of the total cholesterol test card of Hangzhou AllTest Biotechnology Co., Ltd. is relatively high.

**2.6 Time flex****Method**

Test the optimal reading time of the total cholesterol test card. According to the operation steps in the manual, select a batch of total cholesterol test cards and conduct tests with five different

concentrations of whole blood standards. Repeat each test three times. The results were read at 1.5 minutes, 2 minutes, 2.5 minutes and 3 minutes after sample addition respectively. The results are shown in the following table.

**Result:**

Table: Time flex Study Results

Lot	Time(minutes)	Specimen	Concentration range(mg/dL)	Is it within the range		
				Test1	Test2	Test3
ATTCH25020001	1.5	50mg/dL	0-99	no	no	no
		125mg/dL	100-149	no	no	no
		175mg/dL	150-199	no	no	no
		225mg/dL	200-249	no	no	no
		275mg/dL	Above 250	no	no	no
	2	50mg/dL	0-99	yes	yes	yes
		125mg/dL	100-149	yes	yes	yes
		175mg/dL	150-199	yes	yes	yes
		225mg/dL	200-249	yes	yes	yes
		275mg/dL	Above 250	yes	yes	yes
	2.5	50mg/dL	0-99	yes	yes	yes
		125mg/dL	100-149	yes	yes	yes
		175mg/dL	150-199	yes	yes	yes
		225mg/dL	200-249	yes	yes	yes
		275mg/dL	Above 250	yes	yes	yes
	3	50mg/dL	0-99	no	no	no
		125mg/dL	100-149	no	no	no
		175mg/dL	150-199	no	no	no
		225mg/dL	200-249	no	no	no
		275mg/dL	Above 250	no	no	no

**Conclusion:**

The above data shows that the total cholesterol test card displays the correct result within 2 to 2.5 minutes after sample addition. Therefore, we recommend reading the result within 2 to 2.5 minutes and not reading it after 2.5 minutes.

**2.7 Open Pouch Stability**

**Method:**

The Open Pouch stability of the total cholesterol test card (whole blood) was evaluated using samples from three different batches. At room temperature, five concentrations of whole blood standard samples were tested respectively with total cholesterol test cards after 0, 15, 30, 45, 60, 75 and 90 minutes, in triplicate. The test was carried out in accordance with the packaging instructions. The results are listed in the table below.

Table: Time line for Open Pouch Stability Study

Time Temp.	0 minute	15 minutes	30 minutes	45 minutes	60 minutes	75 minutes	90 minutes

25°C	√	√	√	√	√	√	√
------	---	---	---	---	---	---	---

**Results:**

Table: Open Pouch Stability Study Results

Time(minutes)	Specimen	Concentration range(mg/dL)	Is it within the range								
			ATTCH25020001			ATTCH25020002			ATTCH25020003		
0	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
15	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
30	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
45	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
60	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
75	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
90	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes

**Conclusion:** The rapid total cholesterol test card remains stable at 0, 15, 30, 45, 60, 75, and 90 minutes after opening the pouch. It is determined that the shelf life of the product after opening is within 60 minutes after opening.

## 2.8 Accelerate Stability

### Method

Accelerated Stability of the Total Cholesterol Test Card (Whole Blood) was evaluated using samples from three different batches. These were placed in an incubator with the temperature calibrated at 45 °C and 55 °C, and relative humidity (RH) calibrated at 60%. A series of stability tests were performed at 0, 7, 14, 21, 28, 35, 42, 56 days. Tests were conducted using specimens of different concentrations. Testing at each specific time interval consisted of triplicates for each specimen. The tests were performed according to the package insert. Results are presented in tables below.

Arrhenius Formula:

$$\ln K = -E_a/RT + \ln A$$

“K” mean Rate constant

“A” mean Arrhenius constant

“E<sub>a</sub>” mean Activation energy

“R” mean Gas constant

“T” mean Temperature in Kelvin

Table: Time line for Accelerated Stability Study

Day Temp.	0day	7days	14 days	21 days	28 days	35 days	42 days	56 days
45 °C	√	√	√	√	√	√	√	√
55 °C	√	√	√	√	√	×	×	×

### Results:

Table: 45 °C Accelerated Stability Summary

Day	Specimen	Concentration range(mg/dL)	Is it within the range								
			ATTCH25020001			ATTCH25020002			ATTCH25020003		
0	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
7	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
14	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
21	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes

	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
28	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
		50mg/dL	0-99	yes							
35	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
		50mg/dL	0-99	yes							
42	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
		50mg/dL	0-99	yes							
56	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
		50mg/dL	0-99	yes							

Table: 55°C Accelerated Stability Summary

Day	Specimen	Concentration range(mg/dL)	Is it within the range								
			ATTCH25020001			ATTCH25020002			ATTCH25020003		
0	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
7	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
14	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
21	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes

28	50mg/dL	0-99	yes								
	125mg/dL	100-149	yes								
	175mg/dL	150-199	yes								
	225mg/dL	200-249	yes								
	275mg/dL	Above 250	yes								

**Conclusion:** The Total Cholesterol Test Card (Whole Blood) was stable at 45 °C for 56 days and at 55 °C for 28 days. These data were plotted on an Arrhenius Plot and the shelf life of this product was determined to be at least 18 months from the data of manufacture.

## 2.9 Real Time stability

### Method

A real time stability study of transfer lots will be run at the recommended storage temperatures to verify product stability over length of storage time. At the specified time points, five whole blood standards with concentrations of 50, 125, 175, 225 and 275mg/dl respectively were analyzed and tested using total cholesterol test cards in accordance with the quality control test procedure. Each test is repeated five times. The test results are compared with the color card and the results are read.

The following table lists the designated time points for conducting stability tests.

Table: Time line for Real Time Stability Study

Storage Temp	Day	Month						
	0	3	6	9	12	15	18	21
2~30°C	X	X	X	X	X	X	X	X

### Results:

Table: Real Time Stability Summary

Month	Specimen	Concentration range(mg/dL)	Is it within the range								
			ATTCH25020001			ATTCH25020002			ATTCH25020003		
0	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
3	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
6	50mg/dL	0-99	yes	yes	yes	yes	yes	yes	yes	yes	yes
	125mg/dL	100-149	yes	yes	yes	yes	yes	yes	yes	yes	yes
	175mg/dL	150-199	yes	yes	yes	yes	yes	yes	yes	yes	yes
	225mg/dL	200-249	yes	yes	yes	yes	yes	yes	yes	yes	yes
	275mg/dL	Above 250	yes	yes	yes	yes	yes	yes	yes	yes	yes
9	50mg/dL	0-99									
	125mg/dL	100-149									
	175mg/dL	150-199									

	225mg/dL	200-249									
	275mg/dL	Above 250									
12	50mg/dL	0-99									
	125mg/dL	100-149									
	175mg/dL	150-199									
	225mg/dL	200-249									
	275mg/dL	Above 250									
15	50mg/dL	0-99									
	125mg/dL	100-149									
	175mg/dL	150-199									
	225mg/dL	200-249									
	275mg/dL	Above 250									
18	50mg/dL	0-99									
	125mg/dL	100-149									
	175mg/dL	150-199									
	225mg/dL	200-249									
	275mg/dL	Above 250									
21	50mg/dL	0-99									
	125mg/dL	100-149									
	175mg/dL	150-199									
	225mg/dL	200-249									
	275mg/dL	Above 250									

**Conclusion:** The data results show that the total cholesterol test card (whole blood) can be stably stored for 6 months at 25°C, which can confirm that the shelf life of this product is at least 3 months from the production data.

## 2.10 Sample size test

### Methods:

Test the amount of sample (whole blood) required for the total cholesterol test card (whole blood). Five concentrations of whole blood standard samples were tested respectively. Each standard sample was added in three different volumes of 5µL, 10µL, and 15µL respectively. A batch of products was tested in three batches. The test was conducted in accordance with the packaging instructions. The results are listed in the table below.

### Results:

Table: Sample size test Results

lot	Specimen	Concentration range(mg/dL)	Is it within the range								
			Sample addition volume								
			5µL			10µL			15µL		
ATTCH25020001	50mg/dL	0-99	no	no	no	yes	yes	yes	no	no	no
	125mg/dL	100-149	no	no	no	yes	yes	yes	no	no	no
	175mg/dL	150-199	no	no	no	yes	yes	yes	no	no	no
	225mg/dL	200-249	no	no	no	yes	yes	yes	no	no	no
	275mg/dL	Above 250	no	no	no	yes	yes	yes	no	no	no

**Conclusion:** There are differences among the test results of the three sample dosages of 5 $\mu$ L, 10 $\mu$ L and 15 $\mu$ L. To reduce errors, 10 $\mu$ L is the best result.

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Numero: 14603513000  
Data di revisione: 2026-01-09

# Total Cholesterol Test Card (Whole Blood) Package Insert

REF OTCH-404H	English
---------------	---------

A test for the semi-quantitative determination of Total Cholesterol in human finger prick blood.  
For self-testing *in vitro* diagnostic use only.

**【INTENDED USE】**

The Total Cholesterol Test Card is intended for *in-vitro* diagnostic use for the semi-quantitative determination of cholesterol in human finger prick blood. It is suitable for self-testing use.

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**【PRECAUTIONS】**

- For self-testing *in vitro* diagnostic use only.
- The card should remain in the closed packaging bag until use.
- Do not use after the expiration date.
- Do not touch the sample area of the card.
- Discard any discolored or damaged card.
- The used card should be discarded according to local regulations after testing.

**【STORAGE AND STABILITY】**

Store as packaged in the closed packaging bag, either at room temperature or refrigerated (2-30 °C). Keep out of direct sunlight. Cards are stable through the expiration date printed on the package. **DO NOT FREEZE.** Do not use beyond the expiration date.

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**【SPECIMEN COLLECTION AND PREPARATION】**

1. Acceptable specimens include capillary whole blood from the fingertip.
2. Whole blood specimens should be tested immediately after lancing.
3. Fingertip sample must be obtained from a free spontaneous blood flow after the puncture.
4. **Do not collect capillary blood by pressing or squeezing the finger too hard** as this will push fluid from the tissue into the blood and may cause false results.

**【MATERIALS】**

- |             |           |  |               |
|-------------|-----------|--|---------------|
| • Test Card | • Dropper | <b>Materials Provided</b>                  |               |
|             |           | • Package Insert                           | • Lancet      |
|             |           | <b>Materials Required But Not Provided</b> |               |
|             |           |  | • Alcohol Pad |

- Timer

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Wash your hands with soap and rinse with clear warm water.

Bring the pouch to room temperature before opening it. Open the pouch, remove the test card and place it on a clean and level surface. Run the test within one hour and best results will be obtained if the test is performed immediately after opening the foil pouch.

Remove the dropper, lancet and alcohol pad, place them close to the test card.

1. Carefully pull off and dispose the released cap of the lancet.
2. Use the provided alcohol pad to clean the fingertip of the middle or ring finger as the puncture site. Allow to air dry.
3. Press the lancet, on the side from where the cap was extracted; the tip retracts automatically and safely after use. Massage the hand without touching the puncture site by massaging the hand towards the fingertip of the middle or ring finger to be punctured.
4. Keeping the hand down massage the end that was pricked to obtain a blood drop.
5. **Without squeezing the dropper bulb**, put it in contact with the blood. The blood migrates into the line indicated on the capillary dropper.










You may massage again your finger to obtain more blood if the blood does not reach the indicated line. Avoid of air bubbles.

6. Release the blood collected into the sample area of the card by squeezing the dropper bulb.
7. Wait for the blood to be totally dispensed in the well and wait 2 minutes before turning the test card over.

**DO NOT turn over the test card before 2 minutes.**

8. Turn over the test card after 2 minutes.
9. Read the results by rotating the card and matching the closest matching color within 30 seconds.

Do not read the result **after 2 minutes 30 seconds** of applying the blood sample.

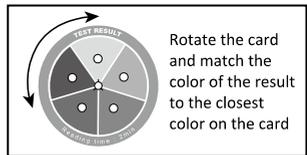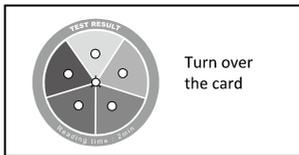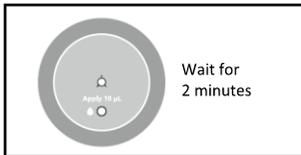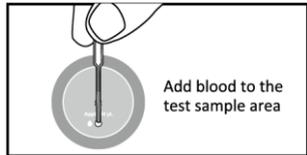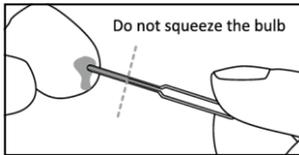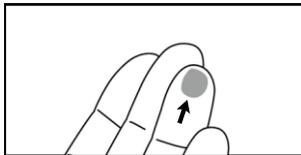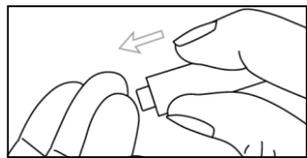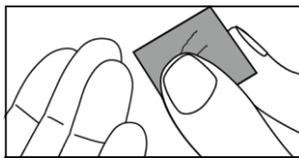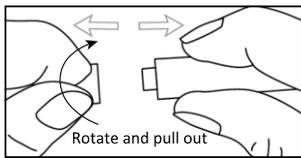### 【TEST RESULT INTERPRETATION】

Range		INTERPRETATION
<b>Normal</b>	0-99 mg/dL (0-2.6 mmol/L) 100-149 mg/dL (2.6-3.9 mmol/L) 150-199 mg/dL(3.9-5.2 mmol/L)	These ranges are considered optimal for total cholesterol. Maintain a healthy lifestyle to keep your levels within this favorable range.
<b>Borderline High</b>	200-249 mg/dL(5.2-6.5 mmol/L)	This is classified as borderline high. It is advisable to monitor your diet and physical activity closely. A comprehensive risk assessment should include evaluation of other blood lipid indicators (such as LDL-C, HDL-C, and triglycerides). Lifestyle modifications are recommended, and medical intervention may be considered based on overall risk.
<b>High</b>	>250 mg/dL(>6.5 mmol/L)	This level is considered high and is associated with increased cardiovascular risk. It is recommended to consult your healthcare provider promptly for a comprehensive evaluation and to discuss appropriate treatment strategies.

### 【INTERFERING SUBSTANCES】

No significant effect was observed at the following concentration for each substance.

Ascorbic acid: 3 mg/dL	Bilirubin 20 mg/dL	Creatinine: 30 mg/dL	Hemoglobin: 400 mg/dL
Urea: 500 mg/dL	EDTA: 640 mg/dL	Heparin-Lithium: 100 mg/dL	Hematocrit: Between 30~55% has no effect on test results.

### Index of Symbols

	Consult instructions for use or consult electronic instructions for use		Contains sufficient for <n> tests		Temperature limit
	In vitro diagnostic medical device		Batch code		Catalogue number
	Caution		Use-by date		Do not re-use
	Do not use if package is damaged and consult instructions for use		Manufacturer		

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**Hangzhou AllTest Biotech Co.,Ltd.**